

QuadroStarPRO^{YELLOW}

Melasma

Dr. Arthur Simon, Jakarta, Indonesia

Before & after 1 month, 1 treatment, scanner 80% coverage, 100% cooling, 18 J/cm², shortest pulse



Melasma

Dr. Atchima Suwanchinda, Thailand

Before & after 17 days, 1 treatment, 1 mm spot, 14 J/cm², defocused brushing technique, shortest pulse



QuadroStarPRO^{YELLOW}

Melasma

Before & after 10 days, 1 treatment, 1 mm spot, scanner 80% coverage at 28 J/cm², 100% cooling, shortest pulse, spot treat brush technique at 16 J/cm², defocused



Pigmented & Vascular Lesions

Dr. Arthur Simon, Jakarta, Indonesia

Before & after 1 month, 1 treatment, 1mm spot, 1 trace, 15 J/cm², shortest pulse



QuadroStarPRO^{YELLOW}

Telangiectasia

Dr. Arthur S. Simon, Jakarta, Indonesia

Before & after 1 month, trace, 1mm spot size, 14-16 J/cm², shortest pulse



Telangiectasia

Dr. Paul Wood, Bad Homburg, Germany

Before & after 3 weeks, scanner 100% coverage, 19 J/cm², 11ms



QuadroStarPRO^{YELLOW}

Telangiectasia

Dr. Paul Wood, Bad Homburg, Germany

Before & after 17 days, 1 mm spot size, 14 J/cm², 18 ms pulse, single pass



Telangiectasia

Dr. Paul Wood, Bad Homburg, Germany



QuadroStarPRO^{YELLOW}

Post-acne Erythema

Dr. Arthur Simon, Jakarta, Indonesia

Before & after 1 month, scanner 80% coverage, 70% cooling, 20 J/cm², shortest pulse



Post-acne Erythema

Dr. Arthur Simon, Jakarta, Indonesia

Before & after 1 month, scanner 80% coverage, 70% cooling, 22 J/cm², shortest pulse



QuadroStarPRO^{YELLOW}

Post-acne Erythema

Dr. Arthur Simon, Jakarta, Indonesia

Before & after 1 month, scanner 80% coverage, 70% cooling, 20 J/cm², shortest pulse + brush 14 J/cm² on spots



Couperose / Rosacea

Dr. Paul Wood, Bad Homburg, Germany

Before & after 3 weeks, 1 treatment, 1mm spot, 12 J/cm², 19ms



QuadroStarPRO YELLOW

Lentigo

Dr. Arthur Simon, Jakarta, Indonesia


Before & after 1 month, brush 12 J/cm² on spot defocused 5 cm, shortest pulse



Wrinkle Reduction

Dr. André Steps, Germany

Before & after 4 months, scanner 100% coverage, 2 passes, 5 treatments, 15-19 J/cm², shortest pulse

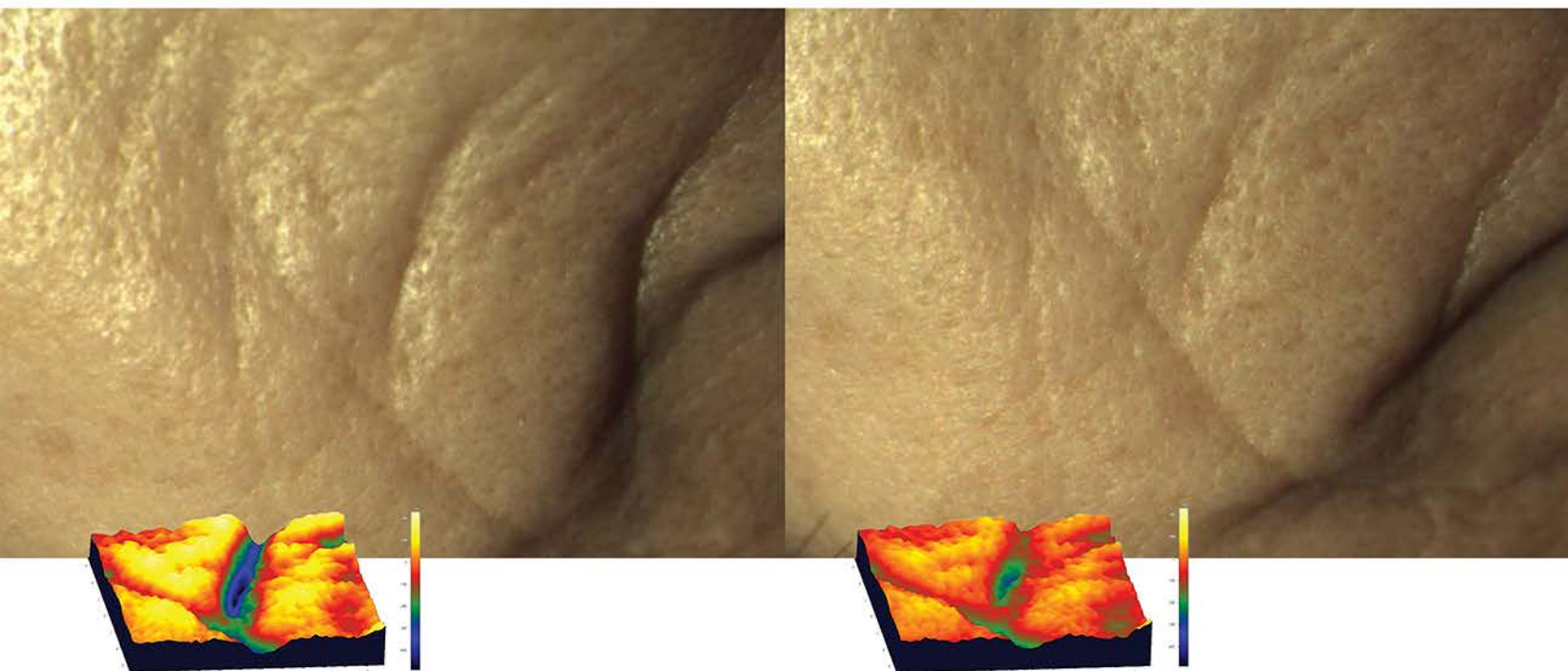

QuadroStarPRO^{YELLOW}

Scar Reduction

Dr. André Steps, Germany

Before & after 38 days, 2 treatments, scanner, 18-23 J/cm²

